Tubes Radiogènes à Anode Tournante
Drehanoden - Röntgenröhre
Tubos de Rayos-X con Ánodo Giratorio

GS-1580/B-180H

Note: Document originally drafted
in the English language.

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

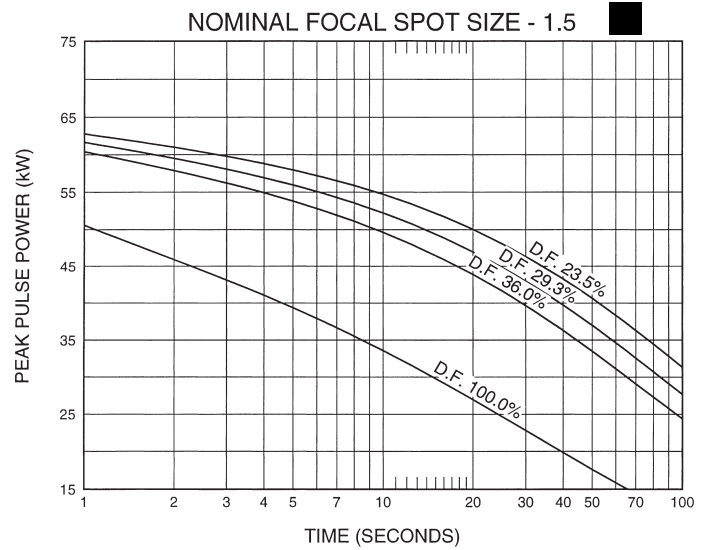
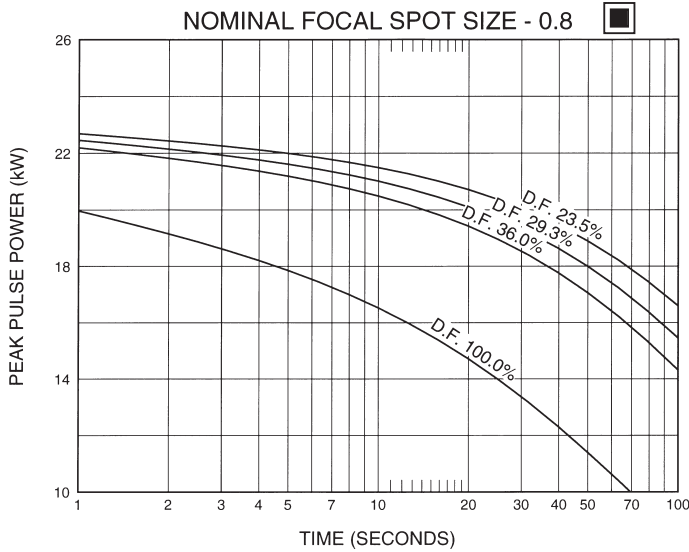
50/60 Hz

150/180 Hz

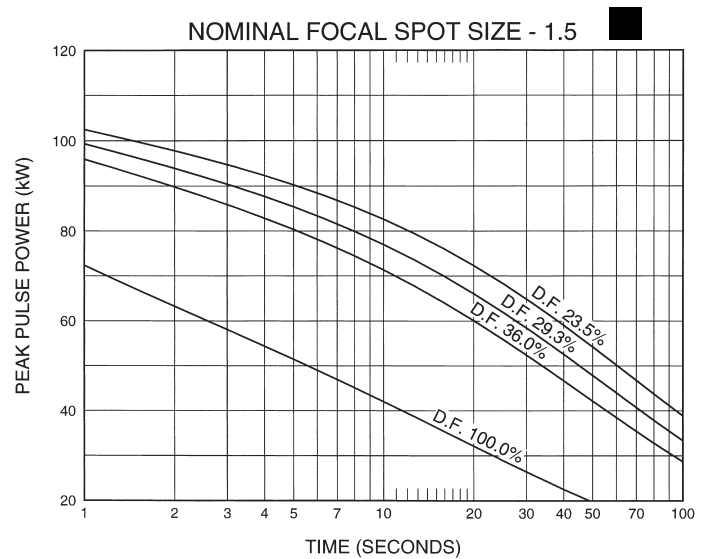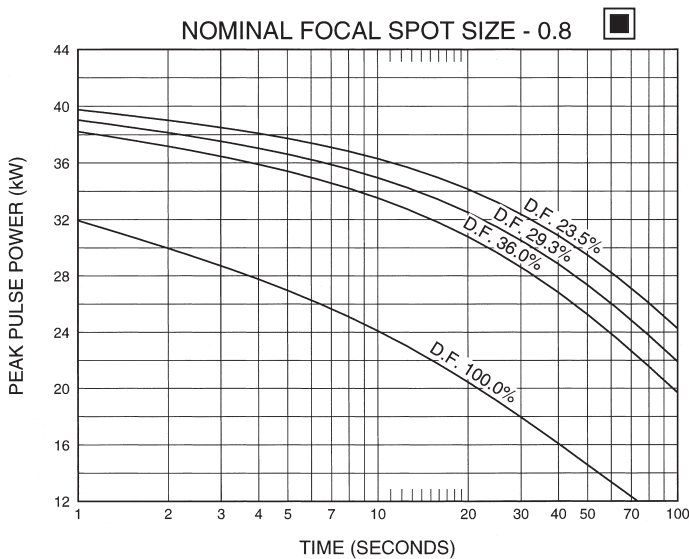Note:
Rating charts reflect maximum tube performance. Tube operation is ultimately limited by system software.

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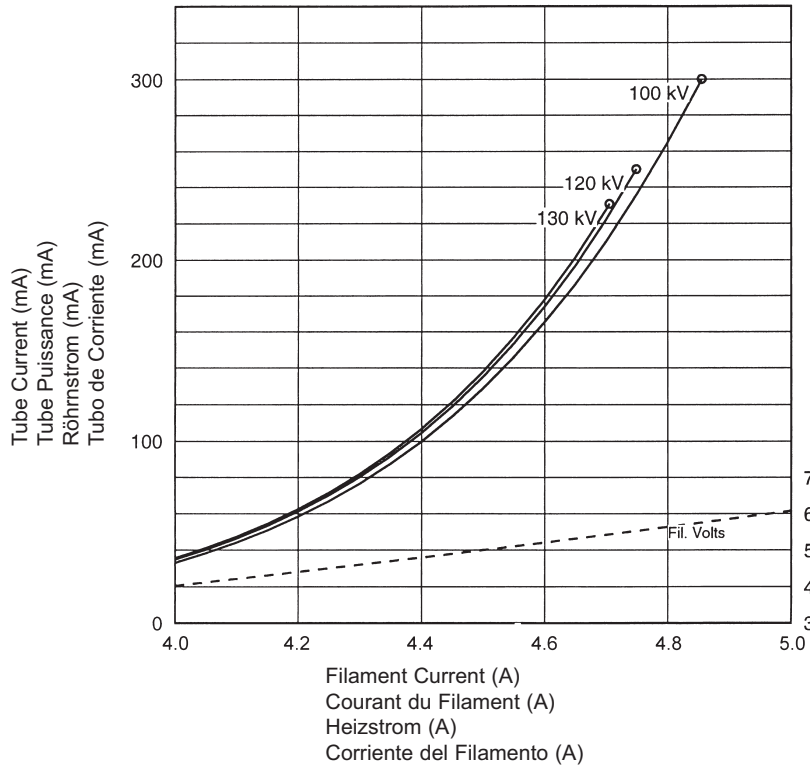
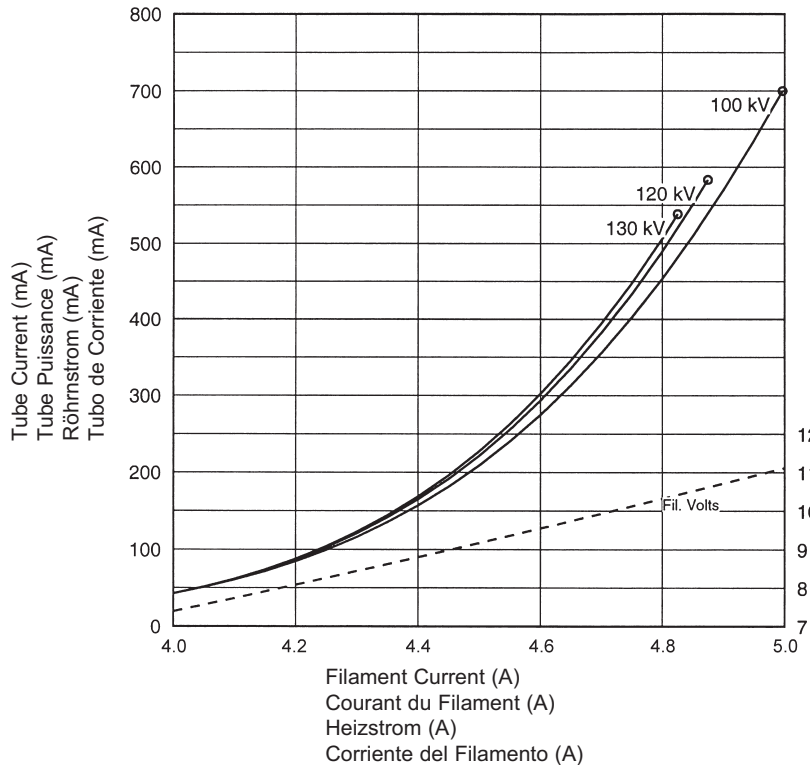
Le Gaine B-180H

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Encaje de B-180H

Maximum Peak Voltage	150 kV
Anode to Ground	75 kV
Cathode to Ground	75 kV
Maximum X-ray Tube Assembly Heat Content	1,430 kJ (2.0 MHU)
Maximum Housing Temperature	78°C
Maximum Continuous Heat Dissipation (Includes stator heat)	2.9 kW (4,060 HU/sec)
Maximum Heat Exchanger Dissipation	5.0 kW (7,050 HU/sec)
Focal Point Position (Central Ray) Within 1mm (X,Y Direction from the center of radiation port.)	
X-Ray Tube Assembly Permanent Filtration	1.0 mm Al IEC 60522
Loading Factors for Leakage Radiation	150 kV, 4 mA
High Voltage Cable Receptacles	Per IEC 60526
Ambient Air Temperature Limits for Operation	5°C to 40°C
Temperature Limits for Storage and Transport	-20°C to +75°C
Humidity	+10% to +90%
Atmospheric Pressure Range	70 kPa to 106 kPa
Weight: Housing & Tube	29.0 kg (63.9 lbs)
Heat Exchanger	16.0 kg (35.3 lbs)
IEC Classification	Class 1
Safety Devices - Thermal Switch Normally Closed Contact	Opening at 76.7°C ±2.8°C (170°F ±5°F)
Rating	3A @ 120Vac or 30 Vdc max.
Flow Switch - Normally Open contact	Contacts close with adequate oil flow
Filament Frequency Limits	0 - 400 Hz
Power Supply	DC

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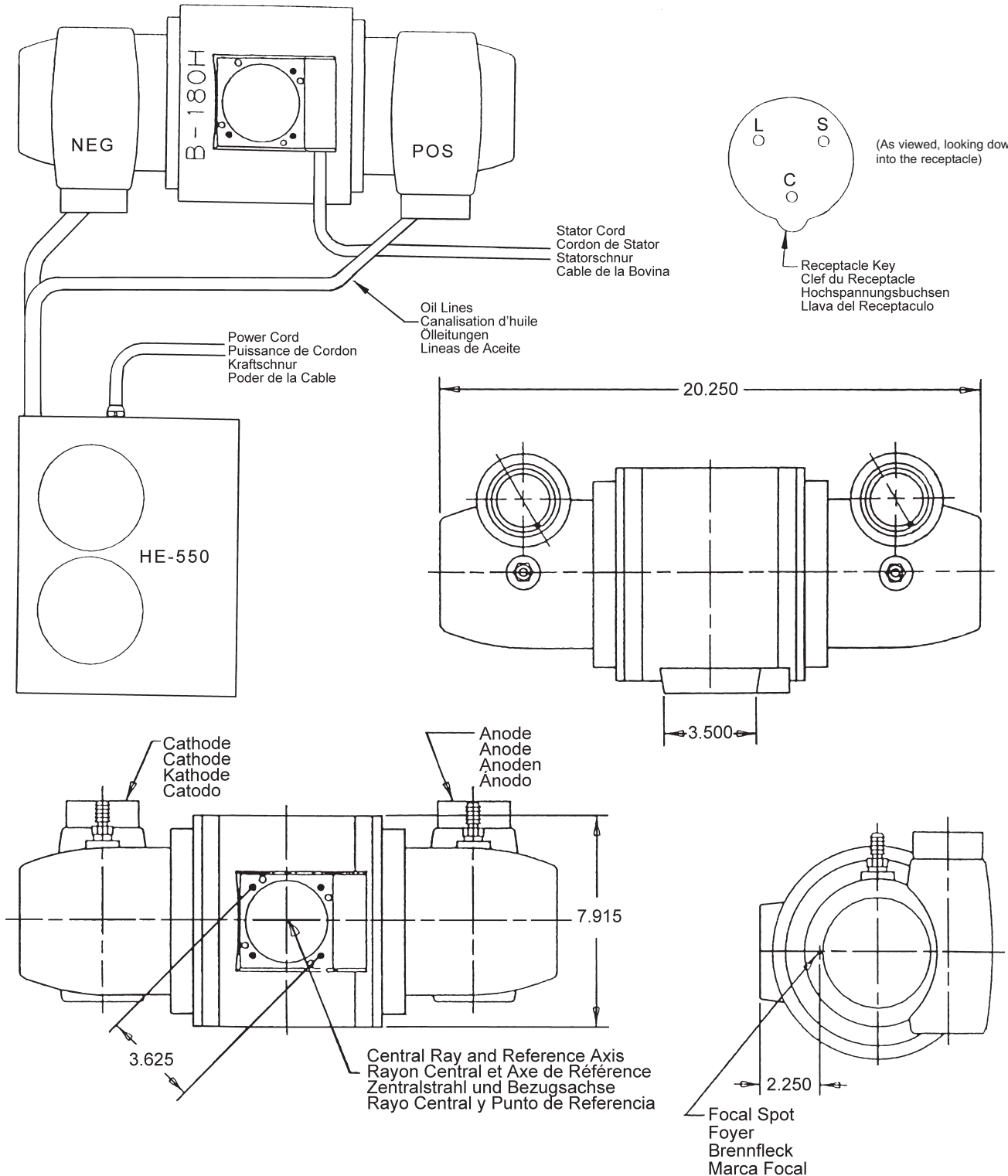
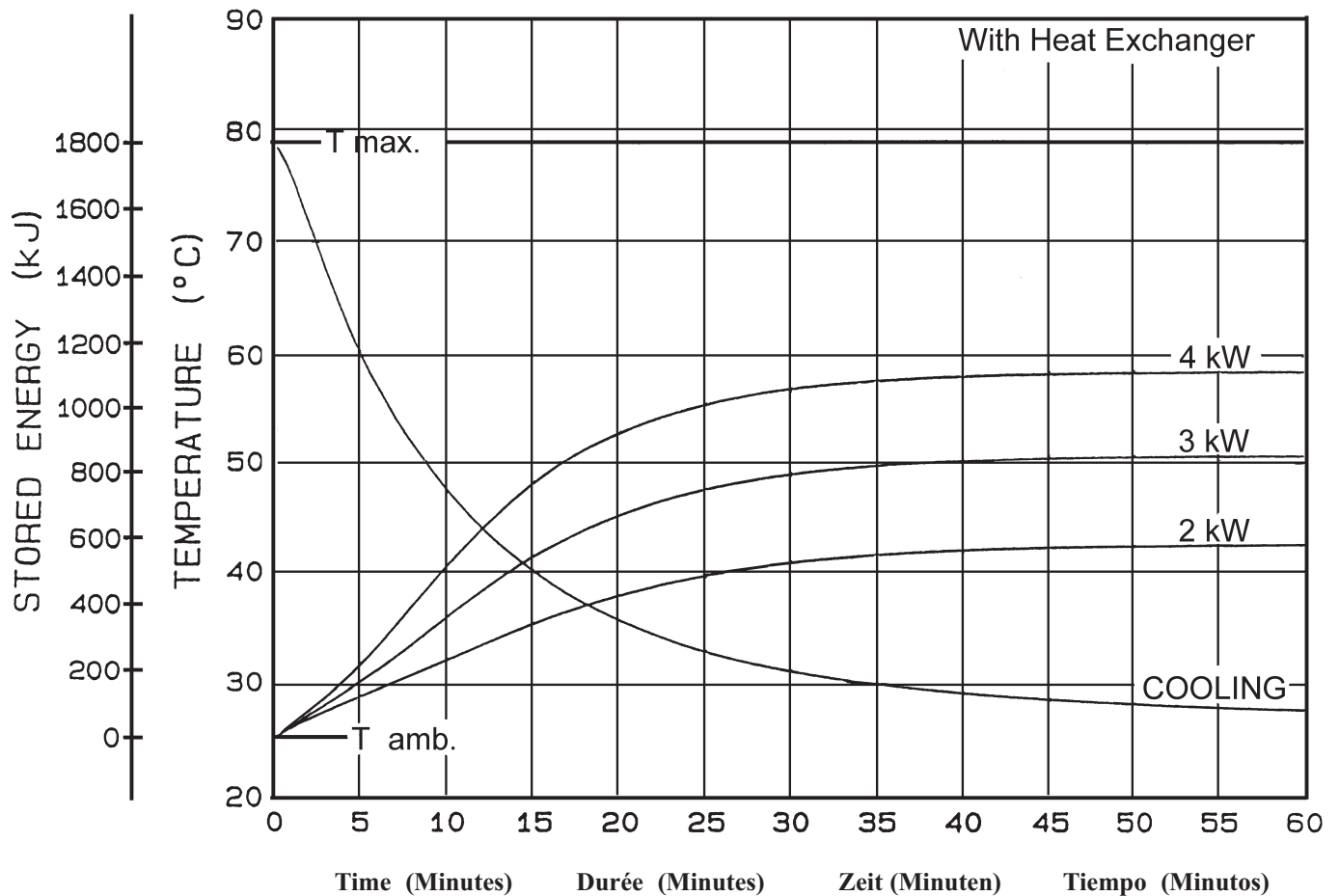
TUBE HOUSING ASSEMBLY HEATING AND COOLING

Note:

- Heat inputs into housing include tube power, filament power, and stator power.
- Heating curves based on no restrictions of natural convection around tube housing assembly.
- Heating and cooling curves reflect maximum tube performance. Tube operation is ultimately limited by system software control.

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
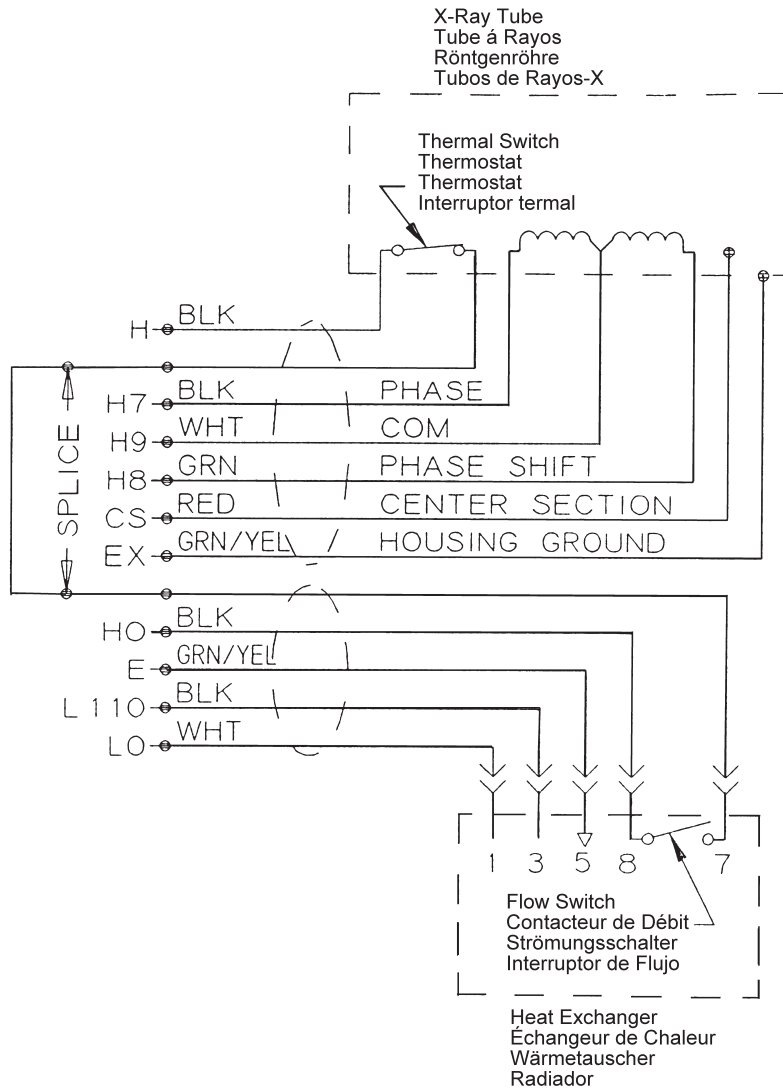
Stator Type: "R" Stator		
Stator Coil Resistance:		
Black to White	14.0 Ohms ±15%	
Green to White	46.0 Ohms ±15%	
Starter Voltage:		
	Start	Run
50/60 Hz	265 VAC	75 VAC
150/180 Hz	600 VAC	115 VAC
Time to Full Speed:		
50/60 Hz	0 - 2700 RPM	6.3 Sec.
150/180 Hz	0 - 8000 RPM	6.3 Sec.
X-Ray Tube Assembly:		
GS-1580/B-180H	IEC 60601-2-28	

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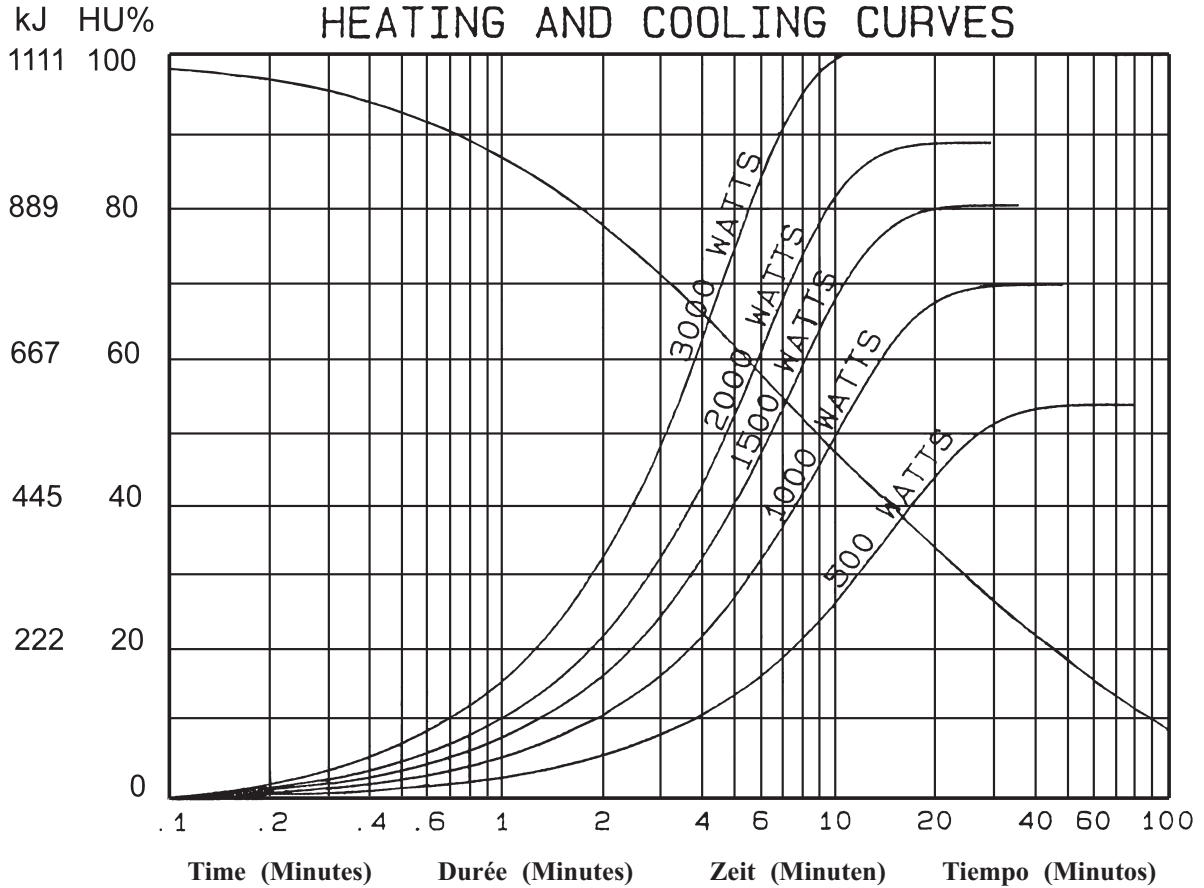Note:
Heating and cooling curves reflect maximum tube performance. Tube operation is ultimately limited by system software control.

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