Tubes Radiogènes à Anode Tournante
 Drehanoden - Röntgenröhre
 Tubos de Rayos-X con Ánodo Giratorio

Note: Document originally drafted in the English language.

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Manufactured by Varian Medical Systems
Fabrique par Varian Medical Systems
Hergestellt von Varian Medical Systems
Fabricado por Varian Medical Systems

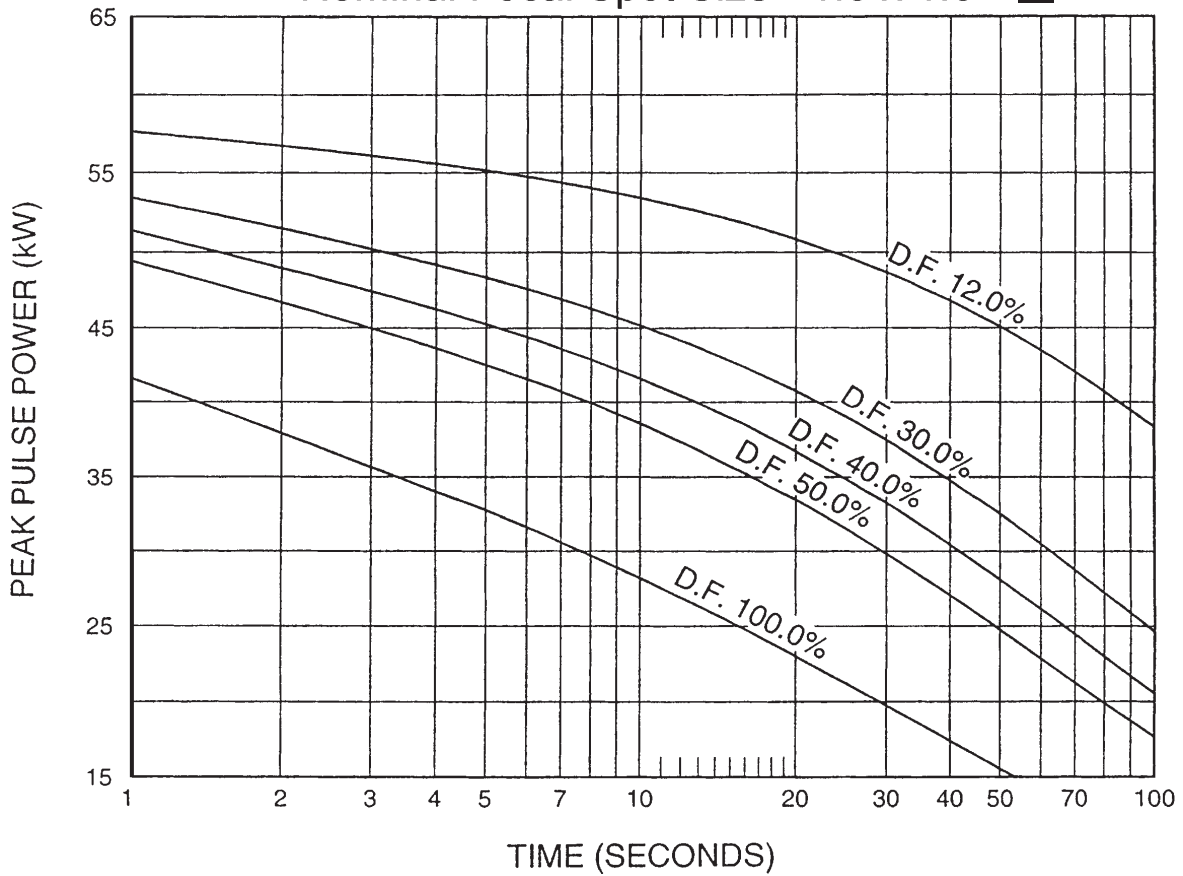
Specifications subject to change without notice.
Spécifications susceptibles d'être modifiées sans préavis.
Technische Daten ohne Gewähr.
Especificaciones sujetas a cambio sin previo aviso.

3 Ø Constant Potential

Abaques de Expositions CEI 60613
Belastungskurven IEC 60613
Diagramas de Exposición IEC 60613

60 Hz

Nominal Focal Spot Size - 1.6 x 1.6

Note:
Rating charts reflect maximum tube performance. Tube operation is ultimately limited by system software.

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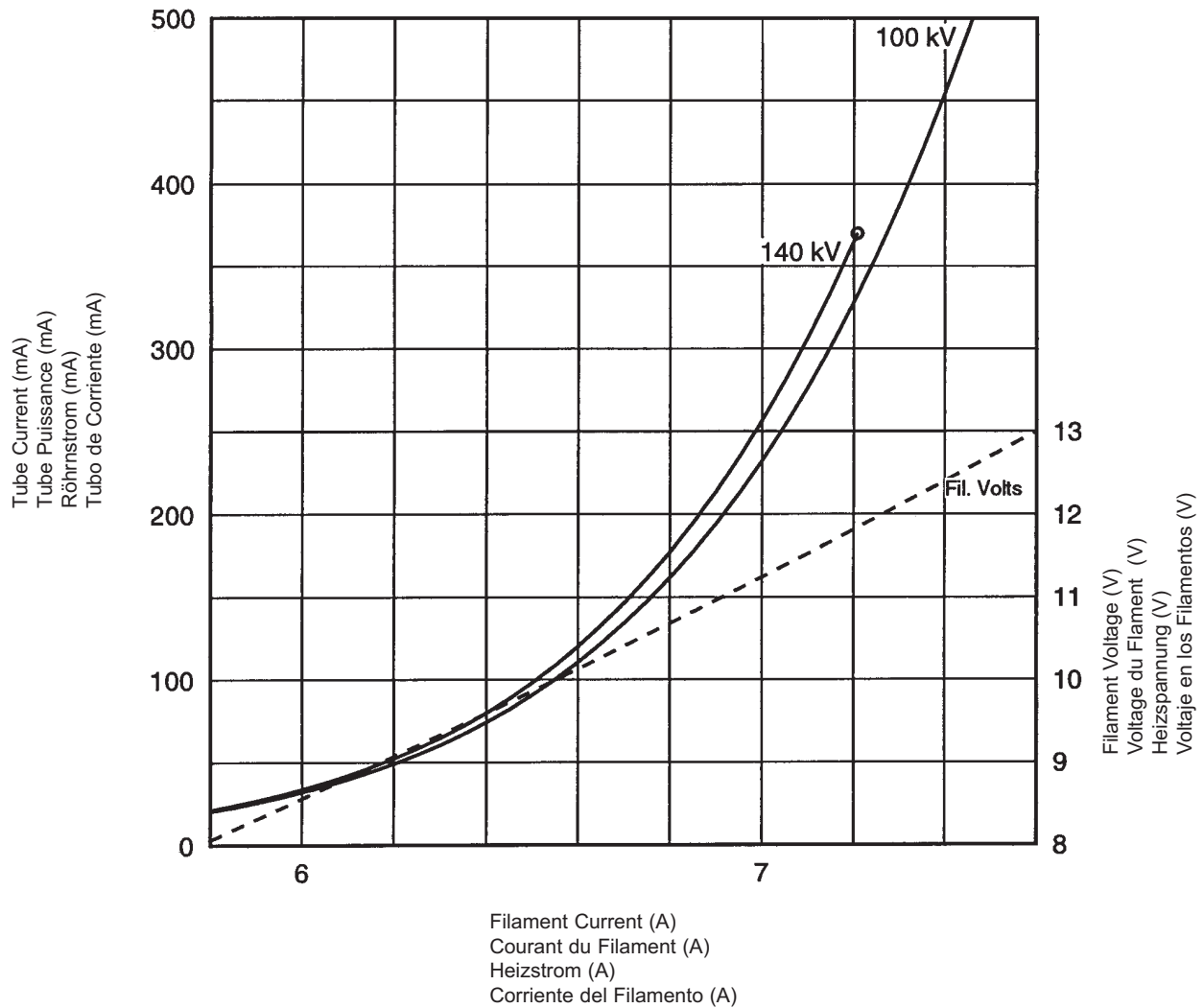
Das B-300H/B-300S Gehäuse

Encaje de B-300H/B-300S

Maximum Peak Voltage	150 kV
Anode to Ground	75 kV
Cathode to Ground	75 kV
Maximum X-ray Tube Assembly Heat Content	1.5 MJ (2.0 MHU)
Maximum Continuous Heat Dissipation (Includes stator heat)	2,900 W (4,060 HU/sec)
Maximum Housing Temperature	78°C
Focal Point Position (Central Ray) Within 1mm (X,Y Direction from the center of radiation port.)	
X-Ray Tube Assembly	
Permanent filtration	2.2 mm Al @ 120 kV IEC 60522
Loading Factors for Leakage Radiation	150 kV, 17 mA
Federal Standard High Voltage Cable	72
Ambient Air Temperature Limits for Operation	5°C to 40°C
Temperature Limits for Storage and Transport	-20°C to +75°C
Humidity	+10% to +90%
Atmospheric Pressure Range	70 kPa to 106 kPa
Weight: Housing	38 kg (83.8 lbs)
IEC Classification	Class 1
Safety Devices - Thermal Switch	
Normally Closed Contact	Opening at 74°C to 80°C
Pressure Switch	
Normally Closed Contact	Opens at 5PSIG ± 3PSIG
Filament Frequency Limits	50 HZ - 20 KHZ
Power Supply	DC
Note: Quick disconnects provided for connection to Scanner cooling unit.	

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

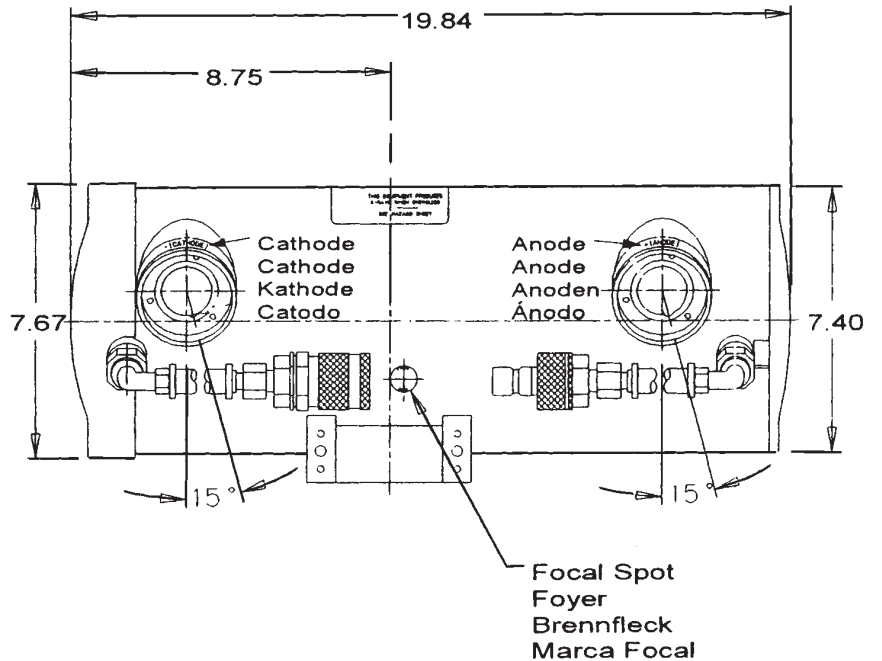
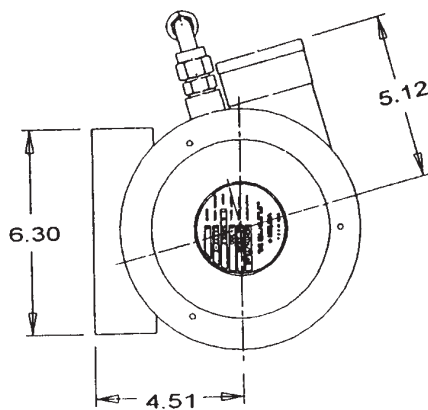
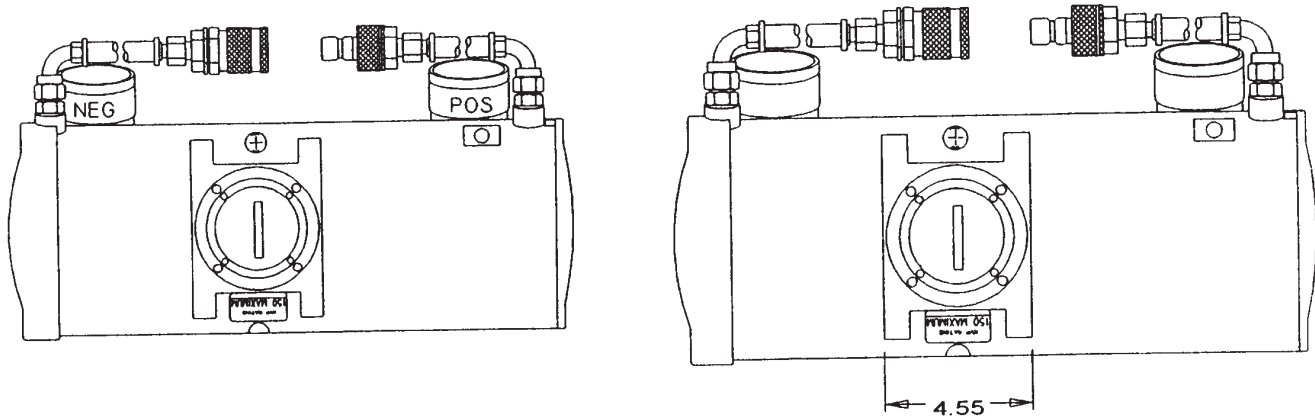
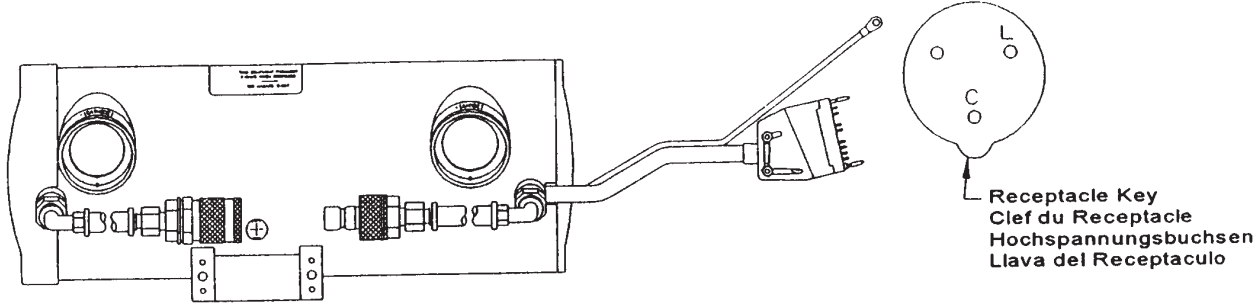
Cable Connections			
Feed Thru Color	J1		P1
	Wire	Terminal	Description
Blue	11	3, 4	Main
Black	0	6, 7	Common
Red	1	9, 10	Phase
Yellow	1a	26, 27	Parallel Phase
White	1b	29, 30	Parallel Phase

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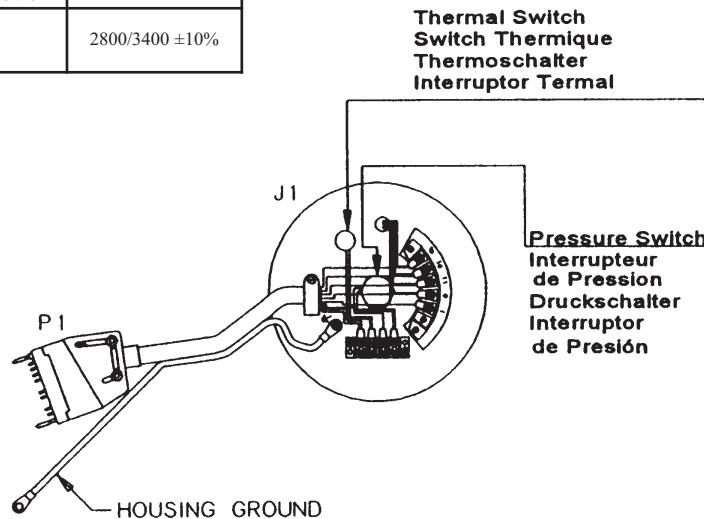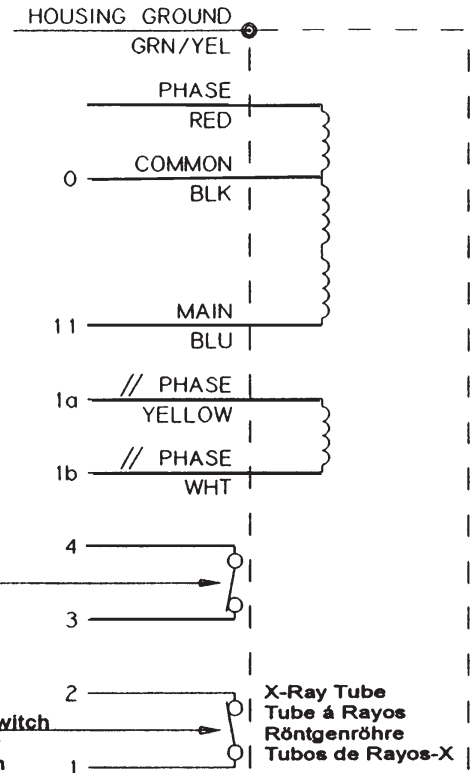
Stator Type:	
Stator Coil Resistance:	
1b to 0	37.0 Ohms ±15%
11 to 0	15.5 Ohms ±15%
Starter Voltage:	
50/60 Hz	Start Run 684 VAC N/A
Time to Full Speed:	
60/50 Hz	0 - 2800 RPM 1.0 Sec.
X-Ray Tube Assembly:	
GS-2030/B-300H	IEC 60601-2-28
GS-2030/B-300S	IEC 60601-2-28

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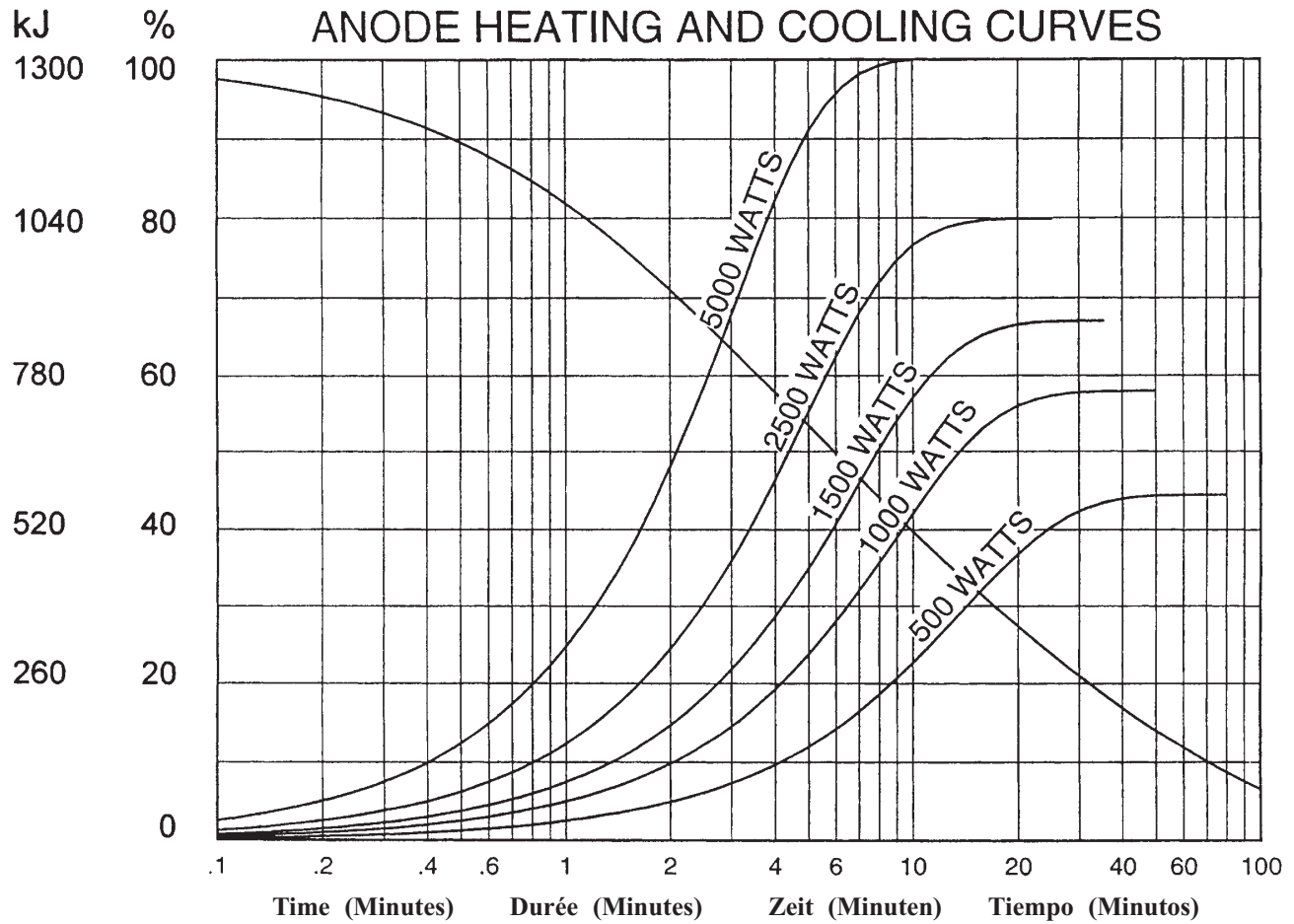Note:

1. Heating and cooling curves reflect maximum tube performance. Tube operation is ultimately limited by system software control.

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