Tubes Radiogènes à Anode Tournante
 Röntgenröhre mit rotierender Anode
 Tubos de Rayos-X con Ánodo Giratorio

Note: Document originally drafted in the English language.

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Manufactured by Varian Medical Systems
Fabrique par Varian Medical Systems
Hergestellt von Varian Medical Systems
Fabricado por Varian Medical Systems

Specifications subject to change without notice.
Spécifications susceptibles d'être modifiées sans préavis.
Technische Daten ohne Gewähr.
Especificaciones sujetas a cambio sin previo aviso.

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Note:
1. Limits are based on maximum track rating except for the following codes:
a - Limited by available heat storage.
b - Limited by window heating.
c - Limited by filament emission.
2. H.S. = Heat Storage
kV = Tube Voltage

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Note:
Rating charts reflect maximum tube performance. Tube operation is ultimately limited by system software.

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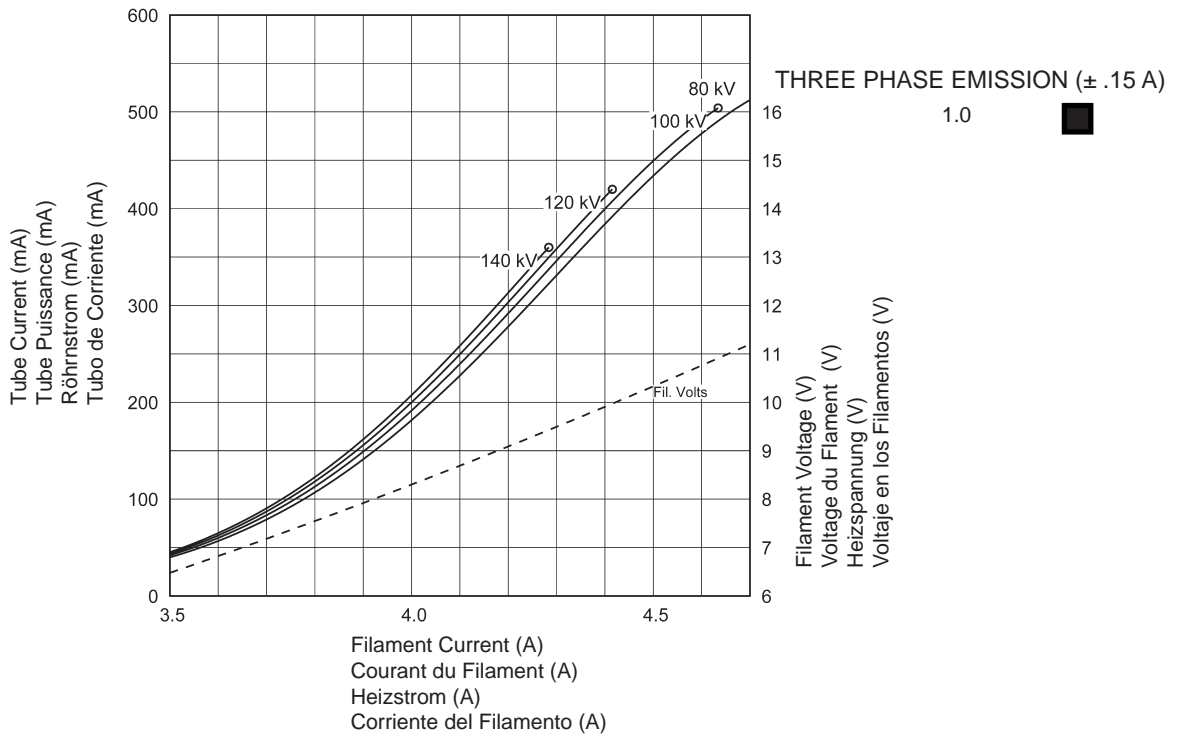
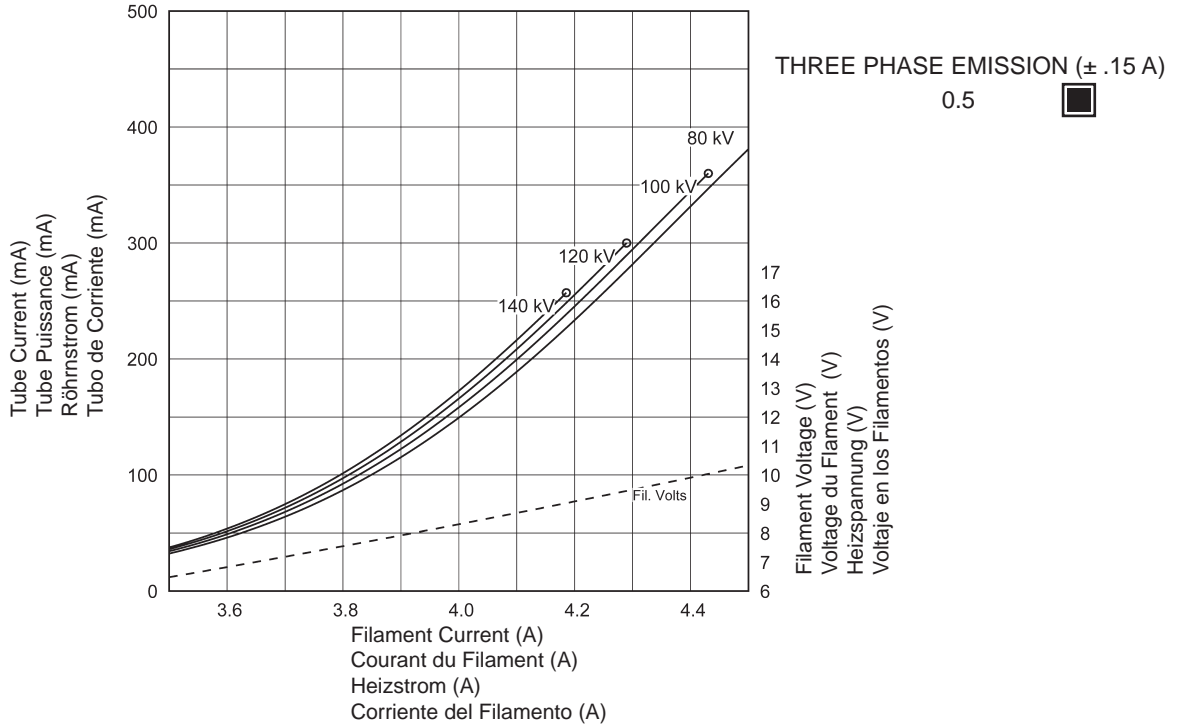
Le Gaine B-320H

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Encaje de B-320H

Maximum Peak Voltage	140 kV
Anode to Ground	70 kV
Cathode to Ground	70 kV
Maximum X-ray Tube Assembly Heat Content	3.9 MJ (5.5 MHU)
Maximum Continuous Heat Dissipation (Includes stator heat)	6.4 kW (9.0 kHU/sec)
Maximum Housing Temperature	80°C
Maximum Heat Exchanger Dissipation	6.4 kW (9.0 kHU/sec)
Focal Point Position (Central Ray) Within 1mm (X, Y Direction from the center of radiation port.)	
X-Ray Tube Assembly	
Permanent Filtration	0.5 mm Al @ 75 kV IEC 60522/1999
Additional Filtration	1.5 mm Al @ 75 kV IEC 60522/1999
Minimum Total Filtration	2.0 mm Al @ 75 kV IEC 60522/1999
Loading Factors for Leakage Radiation	140 kV, 43 mA
High Voltage Cable Receptacles	Per IEC 60526
Ambient Air Temperature Limits for Operation	5°C to 40°C
Temperature Limits for Storage and Transport	-20°C to +75°C
Humidity	+10% to +90%
Atmospheric Pressure Range	70 kPa to 106 kPa
Weight - Housing & Heat Exchanger	214.5 lbs. ±3.5 lbs.
IEC Classification	Class 1
Safety Devices - Thermal Switches	
External - Normally Closed Contact	Opening at 80°C ±3.9°C
Internal - Normally Closed Contact	Opening at 110°C ±4.4°C
Flow Switch - Normally Open contact	
	Contacts close with adequate oil flow.
Filament Frequency Limits	50 HZ - 40 KHZ
Power Supply	DC

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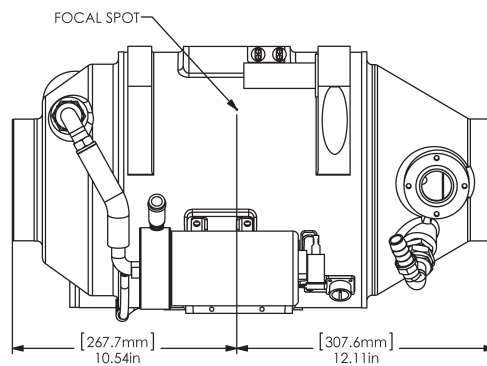
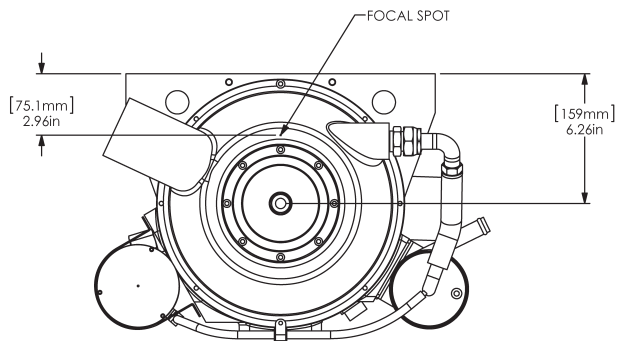
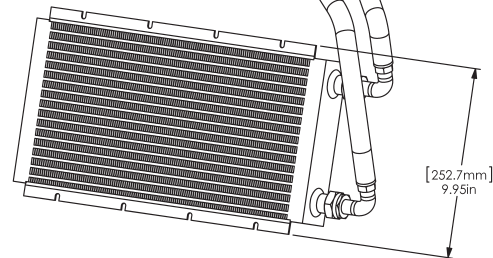
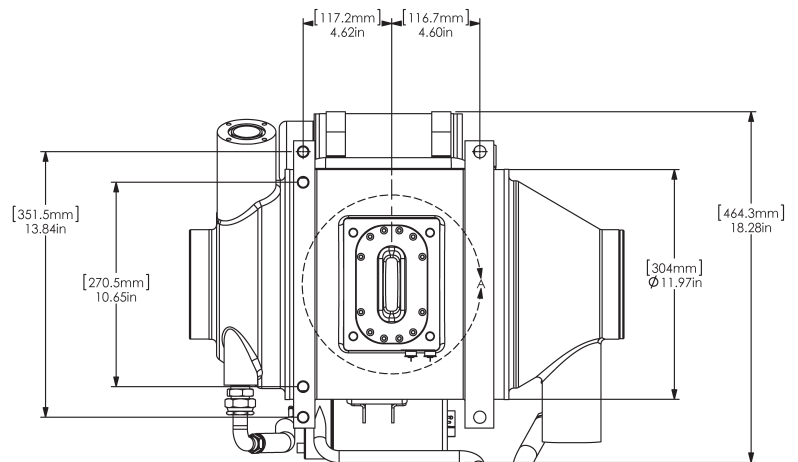
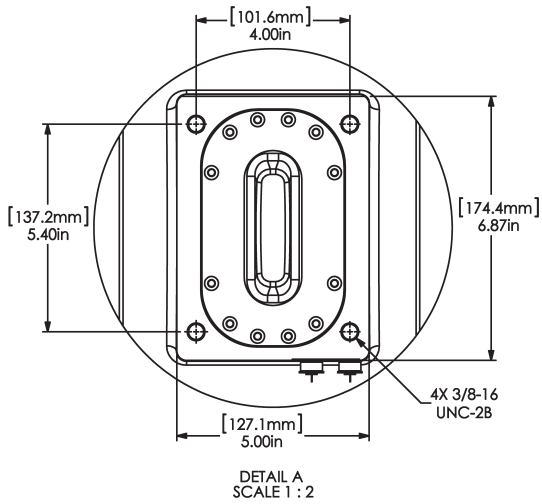
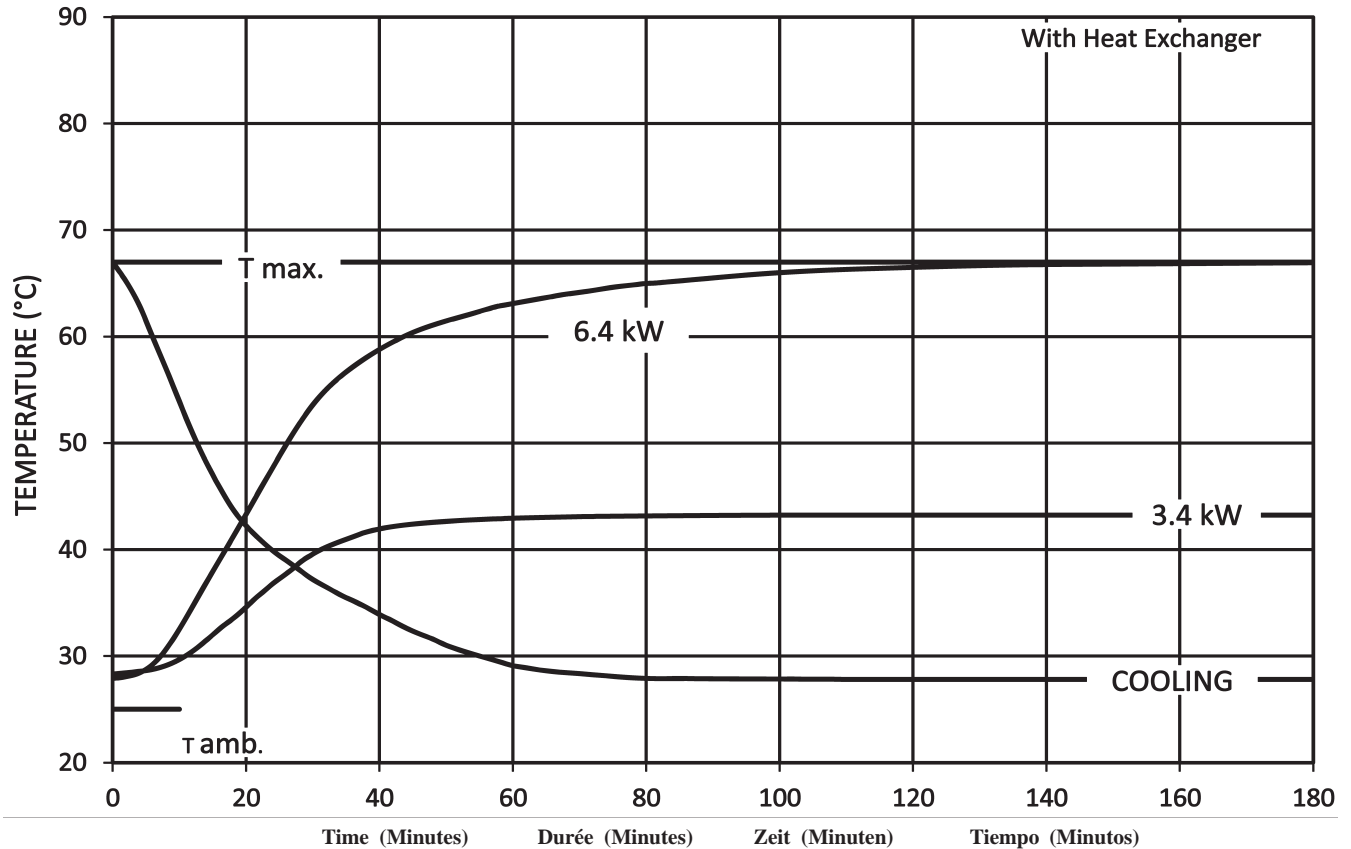
TUBE HOUSING ASSEMBLY HEATING AND COOLING

Note:

- Heat inputs into housing include tube power, filament power, and stator power.
- Heating curves based on no restrictions of natural convection around tube housing assembly.
- Heating and cooling curves reflect maximum tube performance. Tube operation is ultimately limited by system software control.

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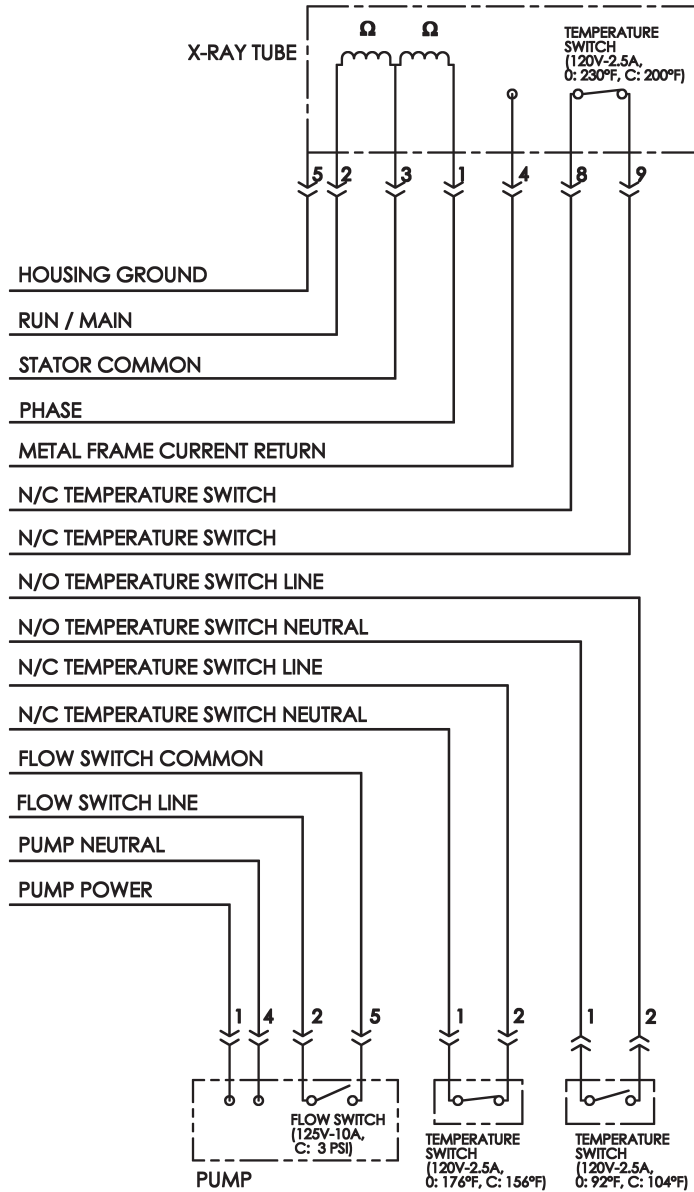
Stator Type:		
Stator Coil Resistance:		
Black to White	8.3 Ω ±15%	
Green to White	8.3 Ω ±15%	
Starter Voltage:		
105 Hz	Start 480 VAC	Run 90 VAC
Time to Full Speed:		
105 Hz	0 - 6300 RPM	10 Sec.
X-Ray Tube Assembly:		
GS-5179/B-320H	IEC 60601-2-28	

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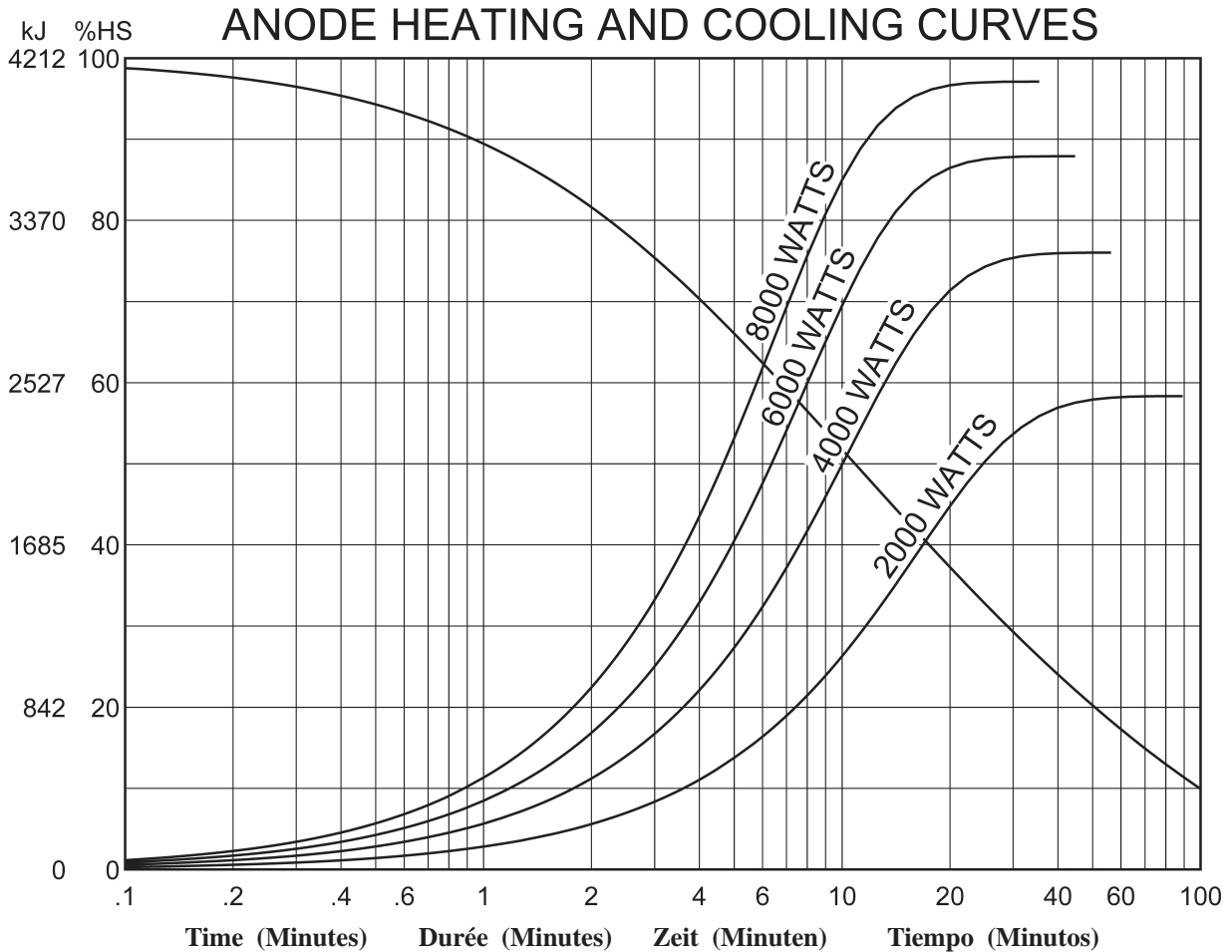
Note:

1. Heating and cooling curves reflect maximum tube performance. Tube operation is ultimately limited by system software control.

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